

HyFoam™

Sterile Hydrophilic Polyurethane Foam Dressing A Synthetic Non-Adherent Absorbent Dressing

Product Description

Hydrophilic foam dressing is a sterile synthetic wound dressing, consisting of hydrophilic absorbent polyurethane foam laminated with a low fouler water proof polyethylene film, which also acts as a bacterial barrier.

HyFoam™ is also offered as a choice with additional adhesive edges for effective anchorage of the dressing over the wound and high patient compliance.

Indication

- Pressure ulcers
- Leg ulcers
- Venous ulcers
- Arterial ulcers
- Diabetic ulcers
- Low to Moderate exuding wounds



SACRUM DRESSING

Application Methodology

- Debride when necessary and irrigate site in accordance with standard protocols.
- Remove excess exudate from surrounding and select a size of the foam dressing that is slightly larger than the wound.
- Apply HyFoam™ in place with a suitable layer to ensure the dressing remains centrally located.
- The foam dressing can remain in situ for up to 7 days (dependent on patient condition and the level of exudates, injury it may be necessary to change the foam dressing whenever good wound care practices dictate that the dressing should be changed).
- Remove the dressing using gentle pressure and remove the dressing from the wound bed and discard.
- If required, cleanse the wound site prior to application of a new dressing.

HyFoam Without adhesive border

5 X 5 CM
10 X 10 CM

AVAILABLE SIZES

HyFoam With adhesive border	Foam Pad Size
10 X 10 CM	5 X 5 CM
15 X 15 CM	10 X 10 CM
17 X 17 CM (Sacrum)	



KRATI
BIOTECH
Healing People

info@kratibiotech.com
kratibiotech@gmail.com
☎ 08-0262880 | 08-0262827
website : www.kratibiotech.com